

READY TO USE FOAM SPRAY FOR CLEANING AND DISINFECTION

Incidin™ Foam

Glucoprotamine based ready-to-use surface cleaning and disinfection foam spray

The ready-to-use surface cleaning and disinfection foam spray combines extremely short contact times with the properties of a gentle cleaner. Through its excellent foam application, it creates an additional reliability for the user.

Incidin Foam also provides:

- ▲ Broad efficacy spectrum
- ▲ Excellent foaming
- ▲ Pleasant odour
- ▲ Compatibility with Incidin Dry Wipes System



Incidin™ Foam

Alcohol based ready-to-use rapid disinfectant foam spray

Why Use Incidin Foam?

BROAD SPECTRUM EFFICACY

Effective against bacteria (incl. TB), yeasts and various viruses, Incidin Foam is also suitable for a wide range of surfaces including high risk areas.

Details on contact times for a wide range of pathogens are in the table below.

CLEANING AND DISINFECTION IN ONE STEP

Providing the efficiency of cleaning and disinfecting in one step, Incidin Foam also offers many other user benefits.

RELIABLE USE

Being ready-to-use, Incidin Foam is very simple to apply and its excellent foaming application guarantees no formation of aerosols.

Due its extensive material compatibility on most surfaces, Incidin Foam gives the assurance of excellent results in a wide variety of application areas.



Incidin Foam
750ml Bottle

Incidin Foam
5L Canister

MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS

EFFICACY UNDILUTED	CONTACT TIME
Disinfection of surfaces acc. to VAH with and without mechanical action (bactericidal, yeasticidal) (clean & dirty conditions)	5 min
Disinfection of surfaces acc. to VAH without mechanical action (tuberculocidal) (clean & dirty conditions)	5 min
Rapid disinfection of surfaces acc. to DGHM (German Society for Hygiene and Microbiology) method 2001 without mechanical action (bactericidal, yeasticidal) (clean & dirty conditions)	1 min
EN 16615 (bactericidal, yeasticidal) (clean & dirty conditions)	5 min
EN 13727 (bactericidal) (clean & dirty conditions)	30 sec
EN 13624 (yeasticidal) (clean & dirty conditions)	30 sec
EN 14348 (tuberculocidal, mycobactericidal) (clean & dirty conditions)	3 min
Effective against enveloped viruses (incl. HIV, HBV, HCV) acc. to RKI recommendation 01/2004*	30 sec
Papova*	15 min
Rota*	5 min

* DVV 2015

EUROPEAN HEADQUARTERS: ECOLAB EUROPE GMBH

Richtstrasse 7
8304 Wallisellen
Switzerland
+41 (0) 44 877 2001
www.ecolab.com

ECOLAB DEUTSCHLAND GmbH

Ecolab-Allee 1
40789 Monheim am Rhein
Germany
+49 (0) 2173 599 0
www.de.ecolab.eu



RAPID. EFFECTIVE. SIMPLE.

APPLICATION AREA

Incidin Foam is a ready-to-use foam spray for cleaning and disinfection of medical devices and all kind of surfaces in healthcare settings.

INSTRUCTIONS FOR USE

Ready-to-use foam spray solution. Spray surface from approx. 30cm distance (max. 40 ml/m²), let product work in and wipe afterwards with a clean, non-woven wipe. Or use product undiluted on a nonwoven wipe, wipe the surface and let dry. Complete wetting of all surfaces to be disinfected is important.

INGREDIENTS

100g contains the following:
20g 2-Propanol, 10g Ethanol,
0.2g Benzalkoniumchlorid, 0.01g
Glucoprotamin®

PACKAGING AND ORDERING INFORMATION

PRODUCT	OUTER SIZE	ORDER CODE
Incidin Foam Bottle	16 x 750ml	XXXXXXX
Incidin Foam Canister	2 x 5L	XXXXXXX
Foam Spray Gun For use with XXXXXXX	5 x 10 pieces	XXXXXXX

STORAGE

Product should be stored in the original container below 25°C in a cool, dry location out of direct sunlight.

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE. FOR PROFESSIONAL USE ONLY.

