

RAPID ALCOHOL DISINFECTION

Incidin® Liquid

Alcohol based ready-to-use surface disinfection spray

This ready to use surface disinfectant combines extremely short contact times with simple handling. The combination of alcohols allows to dry off quickly without leaving critical residues.

In addition, it offers high reliability to the user through its broad spectrum of efficacy including TB and Noro.

Incidin Liquid also provides:

- ▲ Short contact times
- ▲ Aldehyde free formulation
- ▲ Pleasant odour
- ▲ Compatibility with Incidin Dry Wipes System



Incidin® Liquid

Alcohol based ready-to-use surface disinfection spray

Why Use Incidin Liquid?

BROAD EFFICACY SPECTRUM IN SHORT CONTACT TIMES

Due to its broad efficacy spectrum in short contact times, Incidin Liquid is ideal even for intensive care area and other high risk areas. Contact times are detailed in table below.

RELIABLE USE

Being ready-to-use and offering proven material compatibility on alcohol-resistant surfaces, Incidin Liquid ensures the reliable use for all application areas.

ALDEHYDE AND DYE FREE

The aldehyde and dye free formulation offers a pleasant smell and ensures a convenient and reliable product use.



Incidin Liquid
750ml Bottle

Incidin Liquid
5L Canister

MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS

EFFICACY UNDILUTED	CONTACT TIME
Disinfection of surfaces acc. To VAH with/without mechanical action (bactericidal, yeasticidal, tuberculocidal) Clean & Dirty conditions	5 min
Rapid disinfection of surfaces acc. to DGHM (German Society for Hygiene and Microbiology) method 2001 without mechanical action (bactericidal, yeasticidal) (clean & dirty conditions)	1 min
EN 16615 (bactericidal, yeasticidal) (clean & dirty)	1 min
EN 13727 (bactericidal)	1 min
EN 13624 (yeasticidal)	1 min
EN 14348 (tuberculocidal, mycobactericidal)	1 min
Effective against enveloped viruses acc. to RKI recommendation 1/2004 (incl. HIV, HBV, HCV)*	30 sec
Adeno*	1 min
Rota*	30 sec
Papova*	10 min
Noro*	10 min

* DVV 2008

EUROPEAN HEADQUARTERS: ECOLAB EUROPE GMBH

Richtstrasse 7
8304 Wallisellen
Switzerland
+41 (0) 44 877 2001
www.ecolab.com

ECOLAB DEUTSCHLAND GmbH

Ecolab-Allee 1
40789 Monheim am Rhein
Germany
+49 (0) 2173 599 0
www.de.ecolab.eu

CE 0297

FAST. EFFECTIVE. SIMPLE.

APPLICATION AREA

Incidin Liquid is a ready-to-use surface disinfection spray for disinfection of medical devices and all kind of surfaces in healthcare settings.

INSTRUCTIONS FOR USE

Use product undiluted on a non-woven wipe, wipe the alcohol resistant surface and let product work or spray surface from ca. 30cm distance (max. 40ml/m²) and let dry. Complete wetting of all surfaces to be disinfected is important.

INGREDIENTS

100g contains the following:
35g 2-Propanol, 25g 1-Propanol

PACKAGING AND ORDERING INFORMATION

PRODUCT	OUTER SIZE	ORDER CODE
Incidin Liquid Bottle	12 x 650ml	XXXXXXX
Incidin Liquid Bottle	12 x 1L	XXXXXXX
Incidin Liquid Canister	2 x 5L	XXXXXXX

STORAGE

Product should be stored in the original container below 25°C in a cool, dry location out of direct sunlight.

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE. FOR PROFESSIONAL USE ONLY.

ECOLAB®