

COMPLIANCE ENABLING ENVIRONMENTAL DISINFECTION

Incidin® OxyFoam S & OxyWipe S


Sporicidal ready-to-use cleaner and disinfectant with Hi-speed H_2O_2 ™

The innovative formulation of Incidin OxyFoam S & Incidin OxyWipe S provides the benefits of a patented* hydrogen peroxide solution with broad spectrum efficacy.

They are effective against a wide range of bacteria and viruses and leave no active residues due to their rapid evaporation rate.

Incidin OxyFoam S & OxyWipe S provide:

- ▲ Hi-speed H_2O_2 ™
- ▲ EN 16615 compliant
- ▲ Active ingredient decomposes into water and oxygen
- ▲ Sporicidal & virucidal Wipe for all surfaces
- ▲ Excellent material compatibility

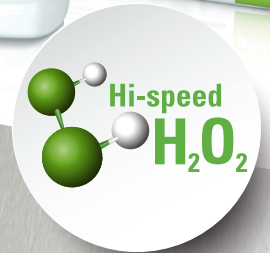

Incidin® OxyFoam S & OxyWipe S

Sporicidal ready-to-use cleaner and disinfectant with Hi-speed H₂O₂™

3 MIN¹

Sporicidal against
C.difficile (SUSPENSIONS TEST)

2 MIN¹

Virucidal
(CLEAN CONDITIONS)

1 MIN¹

Bactericidal
(EN 13727 / 16615)

Features	Benefits
EFFECTIVE	Broad spectrum efficacy Short contact times
SAFE	Active ingredient decomposes into water and oxygen. Excellent material compatibility
CONVENIENT	Cleans & disinfects in one step Ready to use formats

MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS

EFFICACY UNDILUTED	OxyFoam S CONTACT TIME	OxyWipe S CONTACT TIME
Disinfection of surfaces acc. to VAH method 2015 (incl. EN 16615), dirty conditions		
Bactericidal, yeasticidal	5 min	15 min
Fungicidal	5 min	30 min
Mycobactericidal	15 min	30 min
Tested acc. RKI recommendation 01/2004*		
Effective against enveloped viruses acc. to RKI recommendation 01/2004*	15 sec	30 sec
Adeno*	30 sec	1 min
Polyoma SV40*	30 sec	30 sec
Euro-Norms (clean & dirty conditions)		
EN 13727 (bactericidal)	1 min	2 min
EN 13624 (yeasticidal)	1 min	15 min
EN 13624 (fungicidal)	5 min	30 min
EN 14348 (tuberculocidal, mycobactericidal)	5 min	30 min
EN 14476 (virucidal)	2 min** / 30 min	30 min**
EN 14476 Adeno	30 sec	30 sec
EN 14476 Noro	1 min (clean conditions) 15 min (dirty conditions)	15 min
EN 13704 (sporicidal)**	30 min	60 min
EN 13704 (C. difficile)**	3 min	15 min
Euro-Norms (dirty conditions) - 4-field-test		
EN 16615 (bactericidal)	1 min	2 min
EN 16615 (yeasticidal)	1 min	1 min
EN 16615 (fungicidal)	5 min	5 min
EN 16615 (tuberculocidal, mycobactericidal)	5 min	5 min
Sporicidal (method EN 16615)**	15 min	30 min
C. difficile (method EN 16615)**	5 min	5 min

* DVV 2008 ** clean conditions

INCIDIN OXYFOAM S INSTRUCTIONS FOR USE



- 1 Spray surface from ca. 30 cm
- 2 Wipe the surface ensuring it is completely wet
- 3 Let dry



- 1 Apply to clean wipe
- 2 Wipe the surface ensuring it is completely wet
- 3 Let dry

ECOLAB GULF FZE

PO BOX 17063

JAFZA, DUBAI, UAE

Tel. +971 48014000

Email: ecolab.healthcaremea@ecolab.com

en-ae.ecolab.com

CE 0297

EFFECTIVE.
SAFE.
CONVENIENT.

APPLICATION AREA

Incidin OxyFoam S:
Cleaning and disinfection
foam spray for medical
surfaces (incl. e.g. probes)
and inventory.

Incidin OxyWipe S:
Cleaning and disinfection
wipes for medical surfaces
(incl. e.g. probes) and inventory.



CD BIOZID N-12164

INSTRUCTIONS FOR USE

Incidin OxyFoam S: Spray surface from ca. 30 cm or apply to clean wipe. Wipe the surface, ensuring the surface is completely wet for the contact time. Or use product undiluted on a clean wipe, wipe the surface and let dry.

Incidin OxyWipe S: Take single wipe. Wet surface thoroughly, ensuring contact with the entire surface.

Let it dry and do not rinse. Ensure contact with complete surface. Close lid after use. Not for terminal disinfection of invasive medical devices. Do not use on surfaces sensitive to oxidative agents such as marble, copper or brass.

INGREDIENTS

100g contains the following active:

Incidin OxyFoam S: 1.5g

Hydrogen peroxide

Incidin OxyWipe S: 1.5g

Hydrogen peroxide.

PACKAGING AND ORDERING INFORMATION

PRODUCT	OUTER SIZE	ORDER CODE
Incidin OxyFoam S	6 x 750ml	3082060
Incidin OxyFoam S	2x5L	3087450
Incidin OxyWipe S	6 x 100	3082240

STORAGE

Product should be stored in the original container below 25°C in a cool, dry location out of direct sunlight.

Shelf life after first opening:

Incidin OxyFoam S: End of expiry date

Incidin OxyWipe S: One month

References

1. Incidin OxyFoam S

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE. FOR PROFESSIONAL USE ONLY.

ECOLAB®